



# UVC STERILIZATION WARDROBE

## For sanitary personal gowns and clothing

UVC sterilization system (254 nm ). Designed for simultaneous decontamination of sanitary garments.

*What is UVC? According to the NIH are the invisible rays that are parts of the energy that comes from the sun. The ozone layer prevents most UVC radiation from reaching Earth. In the medical field, UVC radiation can also arise from special lamps or a laser beam and is used to remove germs or to help heal wounds.*

### Safe Design

- It is designed to radiate from the 6 interior walls. It has a total of 10 UVC tubes (254 nm).
- Constructed of galvanized steel to better refract radiation.
- Safety system with irradiation lock when opening the door.
- From manufacturing the scheduled time is 30 minutes. (adjustable by technical service between 0 - 30 hr.)



### Simple operation

- Placement of garments to be sterilized.
- Close the door.
- Turn on the unit.
- Press Activation button.
- Wait 30 minutes.
- Open door and remove garments.



### PRINCIPAL OFFICE

José Echegaray, 8 Local 1-3  
28100 – Alcobendas – Madrid - Spain  
Tel. +34 916 624 473 Fax.+34 916 624 469  
[www.tdi.es](http://www.tdi.es) ventas@tdi.es


### DELEGATIONS

Catalunya. Tel. +34 676 351 264  
Galicia. Tel. +34 639 486 955  
Andalucía. Tel. +34 639 481 756

# UVC STERILIZATION WARDROBE

## For sanitary personal gowns and clothing

### Operation

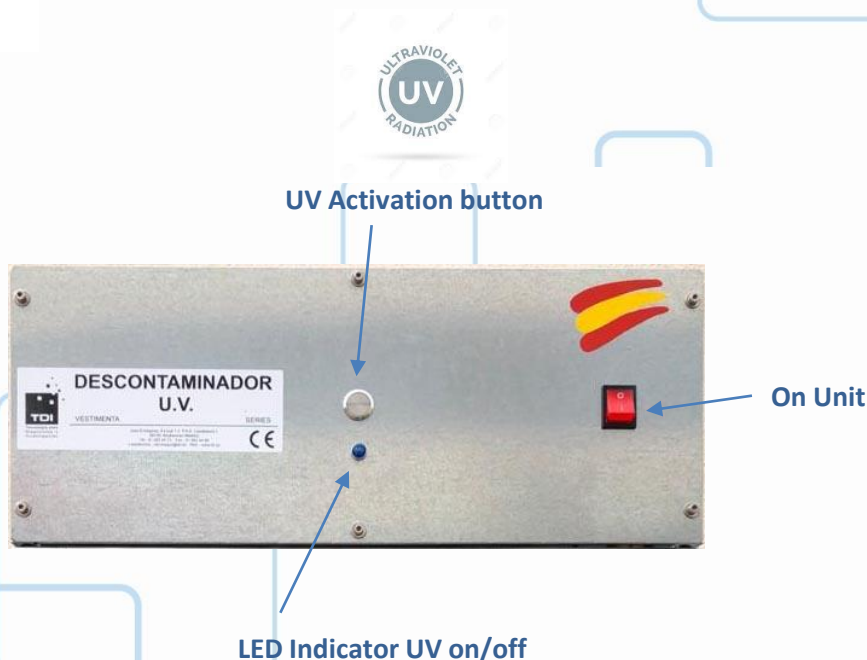

### TECHNICAL ESPECIFICATION

MODEL	WARDROBE UV
Materials	Wardrobe Body: Galvanized steel 1,2 mm.
Wavelength	254 nm
Lamps UVC	2 x 30 w + 8 x 15 w
Radiation time	Scheduled: 30 minutes of manufacturing
Door interlocked	YES, opening it turns off the UV
External Dimensions (AltioxanchoxFondo)	2050 X 500 X 600 mm
Interior Dimensions (AltioxanchoxFondo)	1800 x 495 x 595 mm
Dimensions Packaging (AltioxanchoxFondo)	2100 x 600 x 700 mm
Gross Weight Kg	65 KG
Power	AC 220 V, 50 Hz

MODEL	DESCRIPTION
00-TAQUILLAUUV	UV DECONTAMINATION WARDROBE

### PRINCIPAL OFFICE

José Echegaray, 8 Local 1-3  
28100 – Alcobendas – Madrid - Spain  
Tel. +34 916 624 473 Fax.+34 916 624 469  
[www.tdi.es](http://www.tdi.es) ventas@tdi.es

### DELEGATIONS

Catalunya. Tel. +34 676 351 264  
Galicia. Tel. +34 639 486 955  
Andalucía. Tel. +34 639 481 756